

## 3D Printed Denture Cleaning and Care Guide

Designed and Fabricated by DDS Lab

DDS Lab, one of the largest National Board-Certified full-service dental laboratories in the world, is proud to fabricate the TrueDent™ 3D Printed Denture.

TrueDent 3D Printed Dentures utilize cutting edge 3D printing technology designed to provide you with an improved care and wear experience.

- Choose from a variety of shade profiles for a natural smile.
- Digitally designed and 3D printed precisely for optimal fit.
- Eat and speak with confidence thanks to customized form.
- With TrueDent, your records can be digitally stored and reprinted if needed.

### Caring for Your TrueDent™ Dentures

Your dental professional has prescribed you custom-fit TrueDent dentures. With proper daily care, they will last longer and serve you better. Follow these recommendations to keep your dentures functioning at their best:

When handling dentures outside the mouth, use a folded towel or filled sink to prevent damage if dropped.

1. Remove dentures overnight to rest your gums.
2. Rinse to remove food or denture adhesive residue.
3. Gently brush all surfaces with a denture brush or a soft-bristled toothbrush, focusing especially on the gum area.
4. The denture can be cleaned by the patient with water and a soft brush. After cleaning, the denture should be dried and not soaked in liquid. Abrasive or whitening agents in toothpaste can damage the surface of the denture.
5. Make sure to follow the manufacturer instructions.

### Caution:

- Using bleach, vinegar, boiling water, or stiff brushes can damage dentures. Never place it in the dishwasher. Do not use denture cleansers inside the mouth.

### Prevent Stains:

- Clean dentures immediately after consuming foods such as tomato sauce, red wine, beets, berries, and coffee.
- Avoid tobacco use.
- Treat stains early before they set in and become difficult to remove.

### Checkups & Adjustments

Regular dental appointments are an essential part of your oral health, schedule regular checkups with your dentist every 6 months.

### When to call your doctor

- If your dentures feel uncomfortable, make noise when you talk or eat, appear visibly worn or broken.
- If you develop sores, bleeding gums or bad breath With proper care, your TrueDent dentures will function beautifully.

Contact your dental professional with any additional questions.