

# Reference denture procedure

01

## Patient and order details:

- Fill out the patient's information
- Select denture/removable procedure
- Chart the type of removable
- Due date
- Select restoring lab

02

## Denture details:

- Select stage
- Shade system and teeth shade
- Toggle on which denture to be scanned (*upper, lower, or both*)
- Mould (*determines tooth shape*)
- Gingival shade

03

## Scan options:

- When scanning the denture only, leave denture copy scan toggled off
- Utilize the notes section to communicate specific case instructions with the restoring lab



# Reference denture procedure

04

## Scan mode:

- Scan the internal and external surfaces of the reference denture. When indicated this applies to a singular arch or both arches
- Scan the opposing denture or arch
- Seat the dentures in the patients mouth to scan a bilateral bite.

**Note:** Detailed scanning instruction can be found in PDF titled, "Denture Scanning Guide"

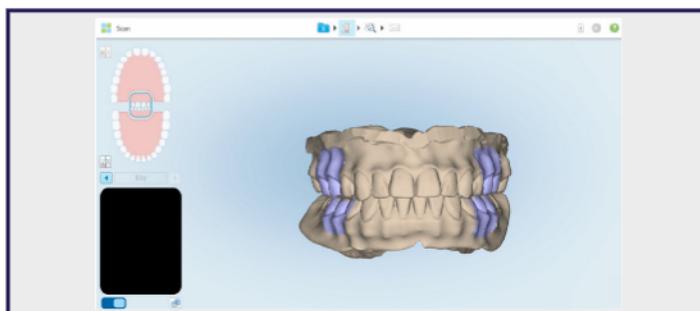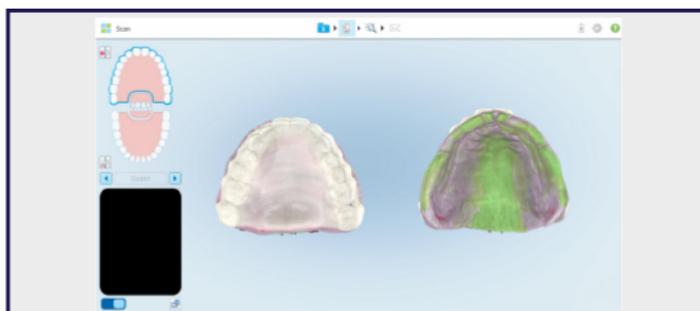

05

## View mode:

- Confirm all denture and edentulous anatomy have been captured.
- Utilize the eraser tool to eliminate a specific region from the image and then rescan it
- Before tapping the envelope to send the scan, tap the Rx icon to confirm all Rx details are complete

**Note:** Adding files to the Rx can be completed from myitero.com before scanning the patient and sending the scan to the restoring dental lab

### Final scan of the reference denture



Contact information