


Copy Denture Procedure

01

Patient and order details:


- Fill out the patient's information
- Select denture/removable procedure
- Chart the type of removable
- Due date
- Select restoring lab

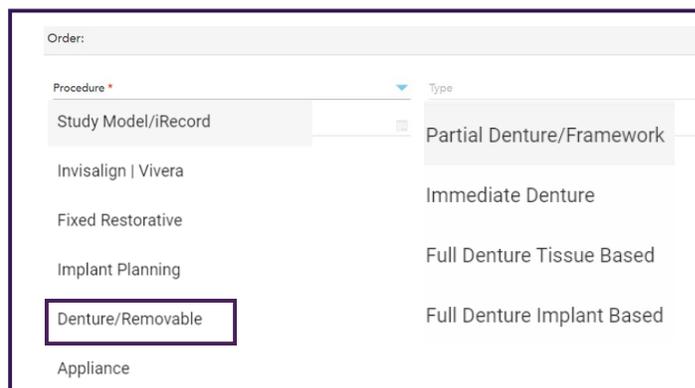

02

Denture details:


- Select stage
- Shade system and teeth shade
- Toggle on which denture to be scanned (*upper, lower, or both*)
- Mould (*determines tooth shape*)
- Gingival shade

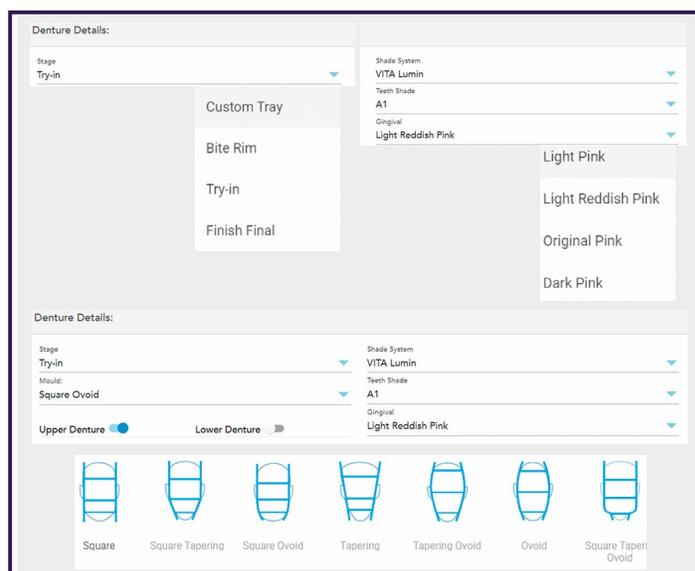

03

Scan options:

- Toggle on denture copy scan to scan the denture and the edentulous arch
- Utilize the notes section to communicate specific case instructions with the restoring lab

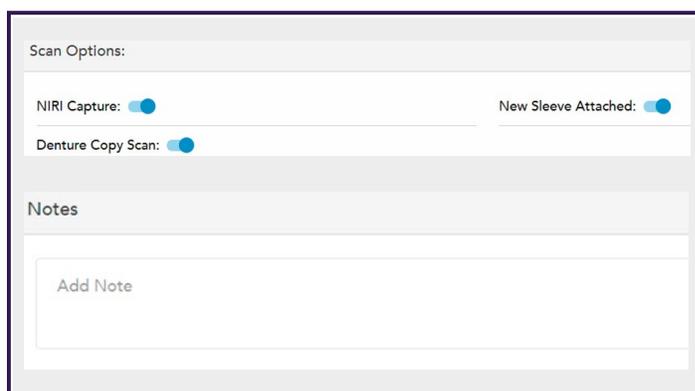



Copy Denture Procedure

04

Scan mode:

- **Scan #1** is indicated for scanning the internal and external surfaces of the copy denture. When indicated this applies to a singular arch or both arches.
- Scan the opposing copy denture or arch.
- Seat the denture(s) in the patient's mouth to scan the patient's bite.
- Tap blue forward arrows to move through segments
- **Scan #2** is indicated for scanning the edentulous arch. When indicated this applies to a singular arch or both arches.

Note: Detailed instructions can be found in PDF titled, "Denture Scanning Guide"

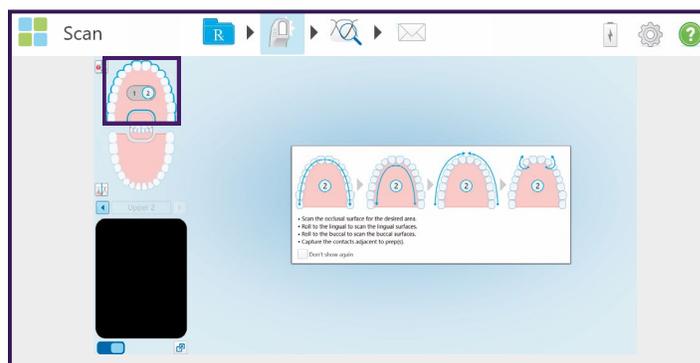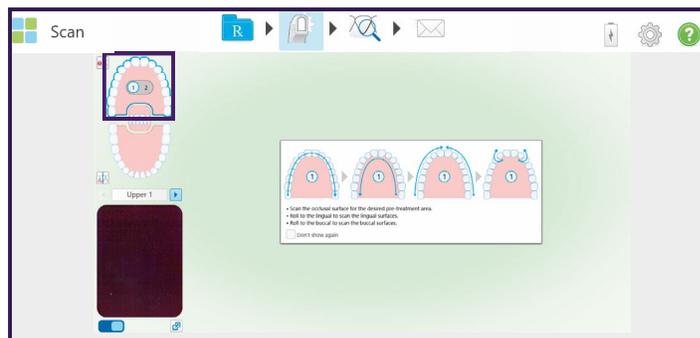


05

View mode:

- Confirm all denture and edentulous anatomy have been captured.
- Use the appropriate eraser tool to correct the scan.
- Before tapping the envelope to send the scan, tap the Rx icon to confirm all Rx details are complete

Final scan of the copy denture

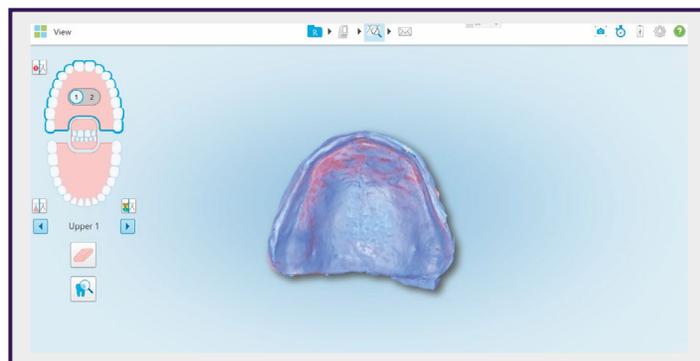

Final scan of the edentulous arch

