

# HOW TO DISINFECT PHYSICAL IMPRESSIONS

01

## WASH IMPRESSIONS

Wearing gloves, wash the impression with water to remove any blood or bioburden saliva that may be remaining on the impression. Use a clean toothbrush to gently scrub the impression.



02

## DISINFECTANT SPRAY

Place the impression on a clean towel. Spray disinfectant around the entire impression and into the undercuts of the impression.



03

## SIT AND RINSE

Let the impression sit for the recommended time period of the disinfectant. Rinse the impression using your hands.



04

## SEND TO LAB

Package the impression in a wet paper towel and plastic bag. Wear gloves at all times while dealing with the impression. Send the impression to the laboratory.

