

DDS Lab

Practice **confidence.**

DIGITAL IMPRESSIONS, WE WORK WITH THEM ALL...



ITERO

- » iTero scanners have **“add preferred lab”**. If DDS Lab is unavailable, please call iTero directly and add DDS as a preferred lab.
- » When the doctor scans any case, the file goes to Align Tech and iTero team will evaluate the scan. If the scan is bad, iTero will contact the doctor. If the scan is good, it takes up to 8 hours before scan is received at DDS Lab.
- » Ortho files must be uploaded through Irecord in the Itero portal. Files will still be accepted through the Itero case managers but that will delay the case.



3 SHAPE TRIOS

- » Trios scanners have **“add connections”**. Send request to cadcam@ddslab.com.
- » We will get the request within 5 minutes. Once approved the doctor will be able to send scans.



SIRONA

- » Sirona scanners have **“add favorite lab”**. The office needs to **add DDS Lab** to their favorite labs.
- » They can send scans right away, they just need to choose DDS Lab from the drop down menu.



CARESTREAM

- » Carestream scanners have “add connections”. Send request to cadcam@ddslab.com or ddslabpix@ddslab.com.
- » Files can be sent through Carestream portal, or upload the STL file through DDS portal.



E4D

- » DDS Labs get E4D scans through **DDX** (Henry Schein portal).
- » Doctor can email the STL file or they can upload through DDS or DDX portals.



Schedule a Consultation
www.ddslab.com



Call DDS Lab directly
(877) 337-7800